**Instructions for use
Breast Shells****Gebrauchsanweisung
Brustwarzenschutz****Mode d'emploi
Coupelles recueil-lait****Istruzioni per l'uso
Proteggi capezzolo****Gebruiksaanwijzing
Tepelbeschermers****Instrucciones de uso
Protectores de pezones****Instruções de utilização
Protetores de mamilo****Εγχειρίδιο χρήσης
Προστατευτικά θηλών**

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The Medela Breast Shells are comfortable to wear. The discreet shape and the flexible silicone membrane, which is gentle on the skin, mean that they are pleasant and invisible to wear. The ventilation holes in the top of the shells allow air to circulate, thus preventing humidity. The breast shell protects sore nipples from further chafing (incorrect positioning of the baby's mouth around the breast could be a reason for the soreness). Milk leakage is simultaneously collected. The shells can be worn continuously between breastfeeding sessions. If soreness becomes severe or persists, consult a physician or lactation consultant. Avoid wearing at night to prevent tender spots.

Assembly and Use

Put the silicone membrane on the shell. Be sure it snaps on all the way around. Place the assembled breast shell inside your bra with the opening in the silicone membrane centered over your nipple. The ventilation holes in the shell must point upwards. We recommend wearing a nursing bra when using a shell; nursing bras are roomier and the shell will fit more comfortably.

CAUTION: Do not feed the breast milk collected in the shell to the baby. Discard this milk.

Cleaning Instructions**Before first use and subsequently once daily**

Place the separated parts in a small pan filled with cold water. Distilled water is recommended – if tap water is used, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale. Bring the water to boil and allow it to continue boiling for approximately 5 minutes. Drain off the water and place the parts on a clean cloth to dry.

After each use

Separate the parts. Shells and membranes should be washed with a mild cleaning solution, rinsed thoroughly in cold water and dried on a clean cloth.

Note

Shells and membranes can be put in a dishwasher.

Material

Shell: Polypropylene / Membrane: Silicone

Disposal

According to local regulations.

Meaning of symbols

REF Indicates the manufacturer's product number (ISO 15223-1, 5.1.6)

LOT Indicates the manufacturer's batch code or lot (ISO 15223-1, 5.1.5)

Fragile (ISO 15223-1, 5.3.1)

Keep dry (ISO 15223-1, 5.3.4)

Protect from sunlight (ISO 15223-1, 5.3.2)

Indicates the need for the user to consult the instructions for use (ISO 15223-1, 5.4.3)

Indicates the date of manufacturing (ISO 15223-1, 5.1.3)

Identifies the manufacturer (ISO 15223-1, 5.1.1)

Indicates that the material is part of a recovery/recycling process

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ACHTUNG: Do not feed the breast milk collected in the shell to the baby. Discard this milk.

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